Fax +32 030 9904733

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Fax +32 030 9904733

Copies of the I/B. Please reproduce them without any changes except under special instruction from Bimar (Italy). The pages must be reproduced and folded in order to obtain a booklet A5 (+/- 148.5 mm width x 210 mm height). When folding, make sure you keep the good numbering when you turn the pages of the I/B. Don't change the page numbering. Keep the language integrity.

This symbol invites you to read these instructions carefully before using the comb, and also inform third parties of them, if necessary. Store this booklet for future reference and for the entire lifetime of the comb. If any parts of these instructions are difficult to understand or if in doubt, contact the company at the address written on the last page before using the product.

WARNING

- This comb is designed only to eliminate fleas and dirt through electronic charges that will not bother your animals, and is exclusively intended for pets (such as dogs, cats or other species of similar size). The fine comb captures impurities and parasites: the dense metal teeth arranged in two rows, have a drop of resin for soft contact with the skin at the ends. Any other use should be considered inappropriate and dangerous. The manufacturer cannot be held responsible for any damages caused by improper, incorrect or irresponsible use and/or by repairs done by an unqualified person. This device must be used in the manner which is described in this manual, and is designed for household use and is not suitable for professional use.
- The comb should be kept away from any sensitive body parts of your pet. Never comb your pet's sensitive areas such as eyes, ears, genitals or anus.
- Check that the coat and the skin of your dog show no signs of matting or injury. Ensure the coat is not wet.
- Only 2 AA batteries (not included) must be used in this appliance. Replacing all batteries at the same time is recommended. Do not mix already used batteries with new batteries. Use new alkaline batteries.
- Should the appliance fail or not work correctly, turn it off. Check that it is not damaged, and thereafter turn it back on: if in any doubt, consult a qualified professional.

Fax +32 030 9904733

Copies of the I/B. Please reproduce them without any changes except under special instruction from Bimar (Italy). The pages must be reproduced and folded in order to obtain a booklet A5 (+/- 148.5 mm width x 210 mm height). When folding, make sure you keep the good numbering when you turn the pages of the I/B. Don't change the page numbering. Keep the language integrity.

- The user must check the appliance periodically so as to not compromise the safety of the pet. Animals should not play with the appliance.
- Do not leave the appliance exposed to atmospheric conditions (sun, rain, etc.).
- Children should not use this comb as a toy.
- After removing the packaging, check the integrity of the appliance; if in doubt, do not use and contact a qualified technician. The packaging materials (plastic bags, polystyrene, metal staples, etc.) must not be left within reach of children or irresponsible persons, as they constitute possible sources of danger and must be disposed of according to applicable regulations.

DESCRIPTION (see pictures)

The appliance is made up of a main unit (1) with:

- place for batteries (2)
 - an On/Off switch (3)
 - dense teeth (4) made of metal. On one end there is a drop of resin for soft contact with the skin.
- The 4cm comb has a protective casing. It is easy to clean with the brush provided.

USING THE APPLIANCE

Before each use, check that the appliance is in good condition, that the comb and its teeth are not damaged: if in doubt consult a qualified professional.

1. Open the cover of the battery slot and insert 2 AA batteries, ensuring the polarities are correct. Close the cover correctly.
2. Press the button to start it (a buzzing sound indicates that the comb is working) and release it to turn it off.
3. The device can operate continuously for a maximum of 5-10 minutes, after which it requires a break of at least 10 minutes before using it again. Failure to do this may affect the performance of the appliance.
4. Use the comb above your pet's skin at an angle of around 45° for best results.
5. When the fleas are caught between the teeth of the comb, the buzzing sound stops temporarily. If the teeth are blocked by fleas, fur or dirt, the buzzing sound is not constant. Clean the teeth with the cleaning brush provided (see figure 1 alongside).
6. After cleaning, the buzzing sound will return and the comb can be used again.

Figure 1

Should the appliance not be used for long periods of time, always remove the batteries.

Practical Advice

- Show the comb to your pets, giving them time to see and smell it.
- It is advisable to use the comb on your pet once or twice a week and this is when you will be able to check for any skin problems, hidden wounds or parasites. Accustom your pet to these hygiene routines as a puppy/kitten.

Fax +32 030 9904733

Copies of the I/B. Please reproduce them without any changes except under special instruction from Bimar (Italy). The pages must be reproduced and folded in order to obtain a booklet A5 (+/- 148.5 mm width x 210 mm height). When folding, make sure you keep the good numbering when you turn the pages of the I/B. Don't change the page numbering. Keep the language integrity.

- It is recommended to use the device before bathing your pet. Make sure the appliance is completely dry and the teeth were cleaned properly.
- Use the comb exclusively when your pet's fur is completely dry. The comb is sensitive to moisture, it does not work on wet fur.
- Separate the fur with your hand until you can see the skin and comb from the skin upwards. Patiently comb zone by zone allowing the comb to run through the pet's fur.
- To avoid irritation by the electric charge, do not touch both sides of the teeth with your fingers (see Figure 2 alongside).
- When fleas are found, use the comb every day for two weeks. Thereafter use the comb 2 or 3 times a week to remove newly hatched fleas.
- The well-being of your pet is most important.

Figure 2

CLEANING AND MAINTENANCE

Warning: before cleaning the comb remove the batteries.

- Comb: requires thorough cleaning after each use to avoid build up of fleas, dandruff, fur, dirt between the teeth. Wash the comb using the brush supplied or a regular small brush with warm water and ordinary dish washing detergent, then rinse with plenty of water, and dry thoroughly.
- The main part of the appliance must only be cleaned with a soft cloth that has been slightly dampened either with just water and/or ordinary detergent. It cannot be washed in the dishwasher, nor under a stream of water: if the electrical parts (battery connections) get wet, dry carefully and completely before using it again.
- If you decide to no longer use the appliance, it is recommended that it is made inoperative by removing the batteries, and that any parts that could represent a risk to playing children are made harmless. Used batteries must be disposed of at appropriate collection centres at your local authority or in suitable containers. In this way, local laws will be respected.

Correctly disposing of the product as per European Directive 2011/65/UE.

At the end of its lifespan, the product must not be disposed of as part of ordinary waste.

It can be handed in to specific collection points set up by the municipality, or to retailers who provide such a service.

Disposing of a household appliance separately avoids possible negative consequences for the environment and human health resulting from inappropriate disposal and allows the recovery of the materials of which it is made in order to obtain significant savings in energy and resources. As a reminder of the obligation to dispose of appliances separately, the product is marked with a wheeled bin symbol. Incorrect disposal of the product by the user will lead to administrative sanctions according to current law.

Fax +32 030 9904733

Copies of the I/B. Please reproduce them without any changes except under special instruction from Bimar (Italy). The pages must be reproduced and folded in order to obtain a booklet A5 (+/- 148.5 mm width x 210 mm height). When folding, make sure you keep the good numbering when you turn the pages of the I/B. Don't change the page numbering. Keep the language integrity.

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